ARCHITECT

SYSTEM

en

Free T₄

REF 7K65

G88321R01

L7K6U9

PLEASE COMMUNICATE TO ALL LABORATORY PERSONNEL

Date Issued April 2017

Product

Product Name	List Number	UDI Number	Lot Numbers	Kit Configuration
ARCHITECT Free T ₄ Reagent Kit	7K65-29	N/A	All until further communication	1 x 100 tests/kit
ARCHITECT Free T ₄ Reagent Kit	7K65-34	N/A	All until further communication	4 x 500 tests/kit
ARCHITECT Free T ₄ Reagent Kit	7K65-39	N/A	All until further communication	1 x 500 tests/kit

Reason

This Product Information letter is being issued to provide you with information regarding a change of measuring interval for the ARCHITECT Free T₄ assay list numbers 7K65-29, 7K65-34 and 7K65-39. Recent data generated by Abbott have determined that the measuring interval is 0.40 ng/dL to 5.00 ng/dL and NOT 0.40 ng/dL to 6.00 ng/dL currently stated in the ARCHITECT Free T₄ package insert. Measuring interval is defined as the range of values in ng/dL which meets the limits of acceptable performance for both imprecision and linearity. A new FT₄ assay file and package insert will be created reflecting the new measuring interval of 0.40 ng/dL to 5.00 ng/dL to be used with list numbers 7K65-29, 7K65-34 and 7K65-39. Availability of the new assay file will be forthcoming.

Current Status

Overall assay performance is not impacted by this change. Only the measuring interval between 5.00 – 6.00 ng/dL is affected. Since the medical decision points are well below 5.0 ng/dL, patient management is not impacted. Calibration range remains unchanged 0.0 to 6.0 ng/dL. Other ARCHITECT Free T₄ assay list numbers 7K65-27, 7K65-32 and 7K65-35 are not impacted by this change.

Action

- Patient sample results between 5.00 ng/dL and 6.00 ng/dL should be reported as >5.00 ng/dL.
 - Please review this letter with your laboratory's management for potential impact specific to your laboratory procedures such as internal ranges.
- If you have forwarded the product listed above to other laboratories, please inform them of this Product Information and provide them a copy of this letter. Please retain this letter for your laboratory records.

Contact

We regret the inconvenience this situation is causing you. If you or any of the health care providers you serve have any questions regarding this information, please contact your local area Customer Service Representative.

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